# ORIGIO® Sequential Fert™

<b>Product No.:</b>	<b>ORIGIO® Sequential Fert™</b>
8301	
8302	0543

<b>Languages:</b>	<b>bg</b>	<b>cs</b>	<b>da</b>	<b>de</b>	<b>el</b>
	<b>en</b>	<b>es</b>	<b>et</b>	<b>fr</b>	<b>hr</b>
	<b>hu</b>	<b>is</b>	<b>it</b>	<b>kk</b>	<b>lt</b>
	<b>lv</b>	<b>mk</b>	<b>nl</b>	<b>no</b>	<b>pt</b>
	<b>ro</b>	<b>ru</b>	<b>sk</b>	<b>sl</b>	<b>sv</b>
	<b>tr</b>				

### Customer Service:

E-mail: customerservice.medicult@origio.com
   
Tel: +45 46 79 02 02 · Fax: +45 46 79 03 02

**ORIGIO a/s**
  
Knardrupvej 2, DK-2760 Måløv, Denmark
   
www.origio.com
   
Tel: +45 46 79 02 00 · Fax: +45 46 79 03 00

# origio

a CooperSurgical Company

#### Ingredientia

**8301 ORIGIO® Sequential Fert™ without phenol red**
  
Acida aminica: Ala, alanylglutaminum, Asn, Asp, Glu, Gly, Pro, Ser, Taurinum; Acidum edeticum; Albumini humani solutio; Aqua; CaCl<sub>2</sub>; Gentamicinum; Glucosum; KCl; Lactas; MgSO<sub>4</sub>; NaCl; NaHCO<sub>3</sub>; NaH<sub>2</sub>PO<sub>4</sub>; Natrii citras; Pyruvas; Vitamini: B<sub>5</sub>, B<sub>9</sub>

**8302 ORIGIO® Sequential Fert™ with phenol red**
  
Acida aminica: Ala, alanylglutaminum, Asn, Asp, Glu, Gly, Pro, Ser, Taurinum; Acidum edeticum; Albumini humani solutio; Aqua; CaCl<sub>2</sub>; Gentamicinum; Glucosum; KCl; Lactas; MgSO<sub>4</sub>; NaCl; NaHCO<sub>3</sub>; NaH<sub>2</sub>PO<sub>4</sub>; Natrii citras; Phenolsulfonpthaleinum; Pyruvas; Vitamini: B<sub>5</sub>, B<sub>9</sub>

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en - english
<p>For fertilization of oocytes by conventional in vitro fertilization (IVF) or Intracytoplasmic Sperm Injection (ICSI).</p>

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This product is for IVF treatment of women, whether the cause of infertility is male or female. The product should only be used by professionals trained in IVF treatment.

**Products**
8301 ORIGIO® Sequential Fert™ without phenol red
8302 ORIGIO® Sequential Fert™ with phenol red

**Contains**
Human albumin solution (HAS)
Gentamicin sulphate 10 µg/ml

**Quality control testing**
Sterility tested (Ph.Eur., USP)
Osmolality tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0.1 EU/ml (Ph.Eur., USP)
HSA analysis (Ph.Eur., USP)
Mouse Embryo Assay (MEA) tested
**Note:** The results of each batch are stated on a Certificate of Analysis, which is available on www.origio.com.

**Storage instructions and stability**
The products are aseptically processed and sup-plied sterile.
Store in original container at 2-8°C, protected from light.
Do not freeze.
Discard excess (unused) media following warning.
The product is to be used within 7 days after opening.
When stored as directed by the manufacturer the product is stable until the expiry date shown on the vial label.

**Precautions and warnings**
Do not use the product if:

- Product packaging appears damaged or if the seal is broken.
- Expiry date has been exceeded.
- The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

**Caution:** All blood products should be treated as potentially infectious. Source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore souce material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can assure assurances that products derived from human blood will not transmit infectious agents.

**Note:** Please note the need for traceability of this product. In addition national legal requirements in your country may exist in this field.

**Note:** Devices used in combination with this device should be intended for the particular purpose.

**Note:** Dispose of the device in accordance with local regulations for disposal of medical devices.

**Instructions for use**

- Equilibrate for a minimum of 2 hours in 5-6% CO₂ at 37°C prior to use. The time required to achieve full equilibration depends on volume of medium and oil overlay.
- Recover oocytes as usual and prepare sperm according to preferred procedure.
- Lleve a cabo la fecundación, ya sea mediante IVF or ICSI, in pre-equilibrated ORIGIO® Sequential Fert™.
- As *ORIGIO® Sequential Fert™ requires CO₂ equilibration, ICSI may also be performed in a HEPES-buffered holding medium. It is recommended that zygotes are transferred to pre-equilibrated culture medium immediately after ICSI.*
- At 16-20 hours post insemination, check for formation of pronuclei, then carefully wash and transfer zygotes to fresh microdrops or wells of pre-equilibrated culture medium.

**Each laboratory should make its own determination of which medium to use for each particular procedure.**

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