ORIGIO® Sequential Cleav™

| | |
|----------------------|---|
| Product No.: |  |
| 8303 8304 |  |
| | 0543 |

| | | | | |
|-----------|-----------|-----------|-----------|-----------|
| bg | cs | da | de | el |
| en | es | et | fr | hr |
| hu | is | it | kk | lt |
| lv | mk | nl | no | pt |
| ro | ru | sk | sl | sv |
| tr | | | | |

Customer Service:

E-mail: customerservice.medicult@origio.com
Tel: +45 46 79 02 02 · Fax: +45 46 79 03 02

ORIGIO a/s
Knardrupvej 2, DK-2760 Måløv, Denmark
www.origio.com
Tel: +45 46 79 02 00 · Fax: +45 46 79 03 00

origio

a CooperSurgical Company

Ingredientia

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| en - english |
|---------------|
| |

For culture until 2-8 cell stage. Can also be used for embryo transfer.

This product is for IVF treatment of women, whether the cause of infertility is male or female. The product should only be used by professionals trained in IVF treatment.

Products
8303 ORIGIO® Sequential Cleav™ without phenol red
8304 ORIGIO® Sequential Cleav™ with phenol red

Contains
Human albumin solution (HAS)
Gentamicin sulphate 10 µg/ml

Quality control testing
Sterility tested (Ph.Eur., USP)
Osmolality tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0,1 EU/ml (Ph.Eur., USP)
HSA analysis (Ph.Eur., USP)
Mouse Embryo Assay (MEA) tested
Note: The results of each batch are stated on a Certificate of Analysis, which is available on www.origio.com.

Storage instructions and stability
The products are aseptically processed and supplied sterile.
Store in original container at 2-8°C, protected from light.
Do not freeze.
Discard excess (unused) media following warming.
The product is to be used within 7 days after opening.
When stored as directed by the manufacturer the product is stable until the expiry date shown on the vial label.

Precautions and warnings
Do not use the product if:
1. Product packaging appears damaged or if the seal is broken.
2. Expiry date has been exceeded.
3. The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Caution: All blood products should be treated as potentially infectious. Source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

Caution: This product contains gentamicin and should not be used on patients that have known allergy to gentamicin or similar antibiotics.

Note: Please note the need for traceability of this product. In addition national legal requirements in your country may exist in this field.

Note: Devices used in combination with this device should be intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instructions for use

- Equilibrate for a minimum of 2 hours in 5-6% CO₂ at 37°C prior to use. The time required to achieve full equilibration depends on volume of medium and oil overlay.
- Transfer zygotes from the fertilization medium to fresh drops or wells of pre-equilibrated ORIGIO® Sequential Cleav™ and return the dish to the incubator.

Embryo transfer

- The embryos are prepared and transferred to the uterus in pre-equilibrated fresh ORIGIO® Sequential Cleav™ or preferred transfer medium.
- Flush the transfer catheter with the chosen transfer medium prior to use.

Each laboratory should make its own determination of which medium to use for each particular procedure.

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