ORIGIO Sequential Blast

Product No.:	1
8305	8306
8306	

Languages:

bg	cs	da	de	el
en	es	et	fr	hr
hu	is	it	kk	lt
lv	mk	nl	no	pt
ro	ru	sk	sl	sv
tr				

Customer Service:

E-mail:

customerservice.medicult@origio.com

Tel: +45 46 79 02 02 · Fax: +45 46 79 03 02

ORIGIO a/s

Knardrupvej 2, DK-2760 Måløv, Denmark

www.origio.com

Tel: +45 46 79 02 00 · Fax: +45 46 79 03 00

origio

a CooperSurgical Company

Ingredientia
8305 ORIGIO Sequential Blast without phenol red
Acida aminica: Ala, alanylglutaminum, Arg, Asp, Asn, Cys, Glu, Gly, His, Ile, Leu, Lys, Met, Phe, Pro, Ser, Thr, Trp, Tyr, Val; Albumini humani solutio; Aqua; Gentamicinum; Glucosum; Hyaluronas; K₂SO₄; Lactas; MgSO₄; NaCl; NaHCO₃; NaH₂PO₄; Pyruvas; SSR; Vitamini: B₁, B₂, B₃, B₇, B₉, Cholinum, Inositolum, Nicotinamidum, Pyridoxinum

8306 ORIGIO Sequential Blast with phenol red
Acida aminica: Ala, alanylglutaminum, Arg, Asp, Asn, Cys, Glu, Gly, His, Ile, Leu, Lys, Met, Phe, Pro, Ser, Thr, Trp, Tyr, Val; Albumini humani solutio; Aqua; Gentamicinum; Glucosum; Hyaluronas; K₂SO₄; Lactas; MgSO₄; NaCl; NaHCO₃; NaH₂PO₄; Phenolsulfonpthaleinum; Pyruvas; SSR; Vitamini: B₁, B₂, B₃, B₇, B₉, Cholinum, Inositolum, Nicotinamidum, Pyridoxinum

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en - english

For culture from the 4-8 cell stage through to the blastocyst stage. Can also be used for embryo transfer.

This product is for IVF treatment of women, whether the cause of infertility is male or female. The product should only be used by professionals trained in IVF treatment.

Products
8305 ORIGIO Sequential Blast without phenol red
8306 ORIGIO Sequential Blast with phenol red

Contains
Human albumin solution (HAS)
Recombinant human insulin
Gentamicin sulphate 10 µg/ml

Quality control testing
Sterility tested (Ph.Eur., USP)
Osmolality tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0.1 EU/ml (Ph.Eur., USP)
HSA analysis (Ph.Eur., USP)
Mouse Embryo Assay (MEA) tested
Note: The results of each batch are stated on a Certificate of Analysis, which is available on www.origio.com.

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Precautions and warnings
Do not use the product if:
1. Product packaging appears damaged or if the seal is broken.
2. Expiry date has been exceeded.
3. The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Caution: All blood products should be treated as potentially infectious. Source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

Caution: This product contains gentamicin and should not be used on patients that have known allergy to gentamicin or similar antibiotics.

Note: Please note the need for traceability of this product. In addition national legal requirements in your country may exist in this field.

Note: Devices used in combination with this device should be intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instructions for use

- Equilibrate for a minimum of 2 hours in 5-6% CO₂ at 37°C prior to use. The time required to achieve full equilibration depends on volume of medium and oil overlay.
- At the 4-8 cell stage the embryos are carefully washed in pre-equilibrated ORIGIO Sequential Blast and transferred to fresh micro drops or wells of ORIGIO Sequential Blast.
- The embryos should be moved to fresh drops/wells of ORIGIO Sequential Blast every other day until blastocyst formation at Day 5/ Day 6.

Embryo transfer

- The embryos are prepared and transferred to the uterus in pre-equilibrated fresh ORIGIO Sequential Blast or preferred transfer medium.
- Flush the transfer catheter with the chosen transfer medium prior to use.

Each laboratory should make its own determination of which medium to use for each particular procedure.

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