CryoSperm™

Product No.:	1101
CE	O543

Languages:

bg	cs	da	de	el
en	es	et	fr	hr
hu	is	it	kk	lt
lv	mk	nl	no	pt
ro	ru	sk	sl	sv
tr	uk			

Customer Service:

E-mail: customer.service@origio.com
Tel.: +45 46 79 02 02 Fax: +45 46 79 03 02

ORIGIO a/s
Knardrupvej 2, DK-2760 Måløv, Denmark
www.origio.com

Tel: +45 46 79 02 00 · Fax: +45 46 79 03 00

origio

a CooperSurgical Company

Ingredientia

1101 CryoSperm™
Aqua; Gentamicinum; Glucosum; Glutaminum; Glycerolum; Glycinum; HEPES; KCl; Lactas; MgSO₄; NaCl; NaHCO₃; NaH₂PO₄; Pyruvas; Raffinosum; Taurinum

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en - english

For freezing of human spermatozoa

This product is for ART treatment, whether the cause of infertility is male or female.
The product should only be used by professionals trained in ART treatment.

Contains
Gentamicin sulphate 10 µg/ml

Quality control testing
Sterility tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0.1 EU/ml (Ph.Eur., USP)
Sperm survival tested
Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.origio.com.

Storage instructions and stability
The products are aseptically processed and supplied sterile.
Store in original container at 2-8 °C, protected from light.
Do not freeze prior to use.
Discard excess (unused) media following warming.
The product is to be used within 7 days after opening.
When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings
Do not use the product if:
1. Product packaging appears damaged or if the seal is broken.
2. Expiry date has been exceeded.
3. The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Note: To avoid cytotoxic effect from dead spermatozoa, etc. it is recommended that the semen sample be prepared soon after collection.

Note: Please note the need for traceability of this product. In addition, national legal requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

- Instructions for use**
Freezing:
1. Pre-warm the CryoSperm™ for a minimum of 2 hours at room temperature.
2. After liquefaction, measure the total volume of the ejaculate and carry out semen analysis as required.
3. Ensure that both the semen sample and the CryoSperm™ are at room temperature and dilute the semen 1:1 (v/v) with the CryoSperm™. The medium should be added drop by drop onto the semen and the solution carefully mixed after each addition.
4. The mixture is left at room temperature for a minimum of 10 minutes.
5. Load the diluted semen into straws or cryo-tubes and seal according to the manufacturer’s recommendations. *It is very important that you leave some air space in the lower part of the straw for sealing and to allow for expansion of the solution during freezing.*
6. Suspend the straws horizontally for 30 minutes, just above the surface of the liquid nitrogen (LN₂). Cryo-tubes should be attached to a cane and then suspended above the surface of the LN₂ for the same period of time.
7. Finally, transfer the straws or cryo-tubes into the LN₂ and store at -196°С.

- Thawing:
1. Warm straws at room temperature for 5 minutes.
2. Open the straws or cryo-tubes according to the manufacturer’s instructions and remove the thawed semen. *Immediately prepare sperm by the density gradient or the swim-up procedure.*

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