Biopsy Medium

Product No.:

1062

Languages:

bg	cs	da	de	el
en	es	et	fr	hr
hu	is	it	kk	lt
lv	mk	nl	no	pt
ro	ru	sk	sl	sv
tr	uk			

Customer Service:

E-mail: customer.service@origio.com
Tel.: +45 46 79 02 02 Fax: +45 46 79 03 02

ORIGIO a/s
Knardrupvej 2, DK-2760 Måløv, Denmark
www.origio.com
Tel.: +45 46 79 02 00 Fax: +45 46 79 03 00

a CooperSurgical Company

Ingredientia

1062 Biopsy Medium
Acida aminica: Ala, Arg, Asp, Asn, Cys, Glu, Gly, His, Ile, Leu, Lys, Met, Phe, Pro, Ser, Thr, Trp, Tyr, Val, Cholinum; Albumini humani solutio; Ammonii molybda; Aqua; Gentamicinum; Glucosum; HEPES; KCl; NaCl; NaHCO₃; Na₂SeO₃; NH₄VO₃; SSR;

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- Take the pre-equilibrated biopsy dish and transfer the embryo into the middle droplet of the dish.
- Perform the drilling of the zona pellucida. *When using Acidified Tyrodes Solution for chemical drilling of the zona, please refer to the specific protocol recommended for the use of this product.*
- Once a small hole in the zona has been obtained, the blastomeres should be readily accessible with the sampling pipette. To minimise the reduction in mass, one or two of the smallest blastomeres are biopsied and placed in the corresponding droplets of Biopsy Medium.

- Immediately after the biopsy, wash the embryo(s) thoroughly by transferring it between several drops/wells of blastocyst culture medium.
- Finally, transfer the embryo into a new dish with pre-equilibrated microdrops/wells of blastocyst culture medium overlaid with pre-equilibrated Liquid Paraffin.
- The biopsy dish with the isolated blastomeres is then ready for sample preparation.

en - english

For blastomere biopsy of cleavage stage embryos for Pre-implantation Genetic Diagnosis (PGD).

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Contains
Human albumin solution (HAS)
Recombinant human insulin
Gentamicin sulphate 10 µg/ml

Quality control testing
Sterility tested (Ph.Eur., USP)
Osmolality tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0.1 EU/ml (Ph.Eur., USP)
HSA analysis (Ph.Eur., USP)
Mouse Embryo Assay (MEA) tested
Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.origio.com.

Storage instructions and stability
The products are aseptically processed and supplied sterile.
Store in original container at 2-8°C, protected from light.
Do not freeze.
Discard excess (unused) media following warming.

The product is to be used within 7 days after opening.
When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings
Do not use the product if:
1. Product packaging appears damaged or if the seal is broken.
2. Expiry date has been exceeded.
3. The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Caution: All blood products should be treated as potentially infectious. The source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products

derived from human blood will not transmit infectious agents.

Note: Subsequent handling of embryos should be gentle so as to reduce the risk of blastomere loss.

Note: Please note the need for traceability of this product. In addition, national legal requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instructions for use
Day 2 (2 days after egg collection)

- Place 1-2 ml of Biopsy Medium in a culture dish, overlay with pre-equilibrated Liquid Paraffin and warm to 37°C in a 5-6% CO₂ environment for a minimum of 2 hours.

Day 3 (day of embryo biopsy)

- Cleavage stage embryo biopsy is carried out early in the morning of Day 3 post-fertilisation.
- Half an hour before the biopsy, prepare a biopsy dish for each embryo. Flush the tip of the pipette with 10 x 10 µl Biopsy Medium. Pipette three droplets of Biopsy Medium and one droplet of Acidified Tyrodes Solution as shown in the illustration.
- Cover the dish immediately with 4 ml of pre-equilibrated Liquid Paraffin to avoid evaporation, and warm the prepared dishes to 37°C in a 5-6% CO₂ environment. At the same time, prepare microdrops/wells of blastocyst culture media, for the wash and final culture of the embryos whilst the diagnosis is carried out.

- Take the pre-equilibrated biopsy dish and transfer the embryo into the middle droplet of the dish.
- Perform the drilling of the zona pellucida. *When using Acidified Tyrodes Solution for chemical drilling of the zona, please refer to the specific protocol recommended for the use of this product.*
- Once a small hole in the zona has been obtained, the blastomeres should be readily accessible with the sampling pipette. To minimise the reduction in mass, one or two of the smallest blastomeres are biopsied and placed in the corresponding droplets of Biopsy Medium.

- Immediately after the biopsy, wash the embryo(s) thoroughly by transferring it between several drops/wells of blastocyst culture medium.
- Finally, transfer the embryo into a new dish with pre-equilibrated microdrops/wells of blastocyst culture medium overlaid with pre-equilibrated Liquid Paraffin.
- The biopsy dish with the isolated blastomeres is then ready for sample preparation.

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