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en - english

For fertilisation of oocytes by conventional in vitro fertilisation (IVF) or Intracytoplasmic Sperm Injection (ICSI).

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Products
8301 ORIGIO® Sequential Fert™
8302 ORIGIO® Sequential Fert™ with phenol red

Contains
Human albumin solution (HAS)
Gentamicin sulphate 10 µg/ml

Quality control testing
Sterility tested (Ph.Eur., USP)
Osmolality tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0,1 EU/ml (Ph.Eur., USP)
HSA analysis (Ph.Eur., USP)
Mouse Embryo Assay (MEA) tested
Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.origio.com.

Storage instructions and stability
The products are aseptically processed and supplied sterile.
Store in original container at 2-8°C, protected from light.
Do not freeze.
Discard excess (unused) media following warming.
The product is to be used within 7 days after opening.
When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings
Do not use the product if:
1. Product packaging appears damaged or if the seal is broken.
2. Expiry date has been exceeded.
3. The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Caution: All blood products should be treated as potentially infectious. The source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/-2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

Note: Please note the need for traceability of this product. In addition, national legal requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instructions for use

- Equilibrate for a minimum of 2 hours in 5-6% CO₂ at 37°C prior to use. The time required to achieve full equilibration depends on the volume of medium and oil overlay.
- Recover oocytes as usual and prepare sperm according to preferred procedure.
- Carry out fertilisation – either conventional IVF or ICSI – in pre-equilibrated ORIGIO® Sequential Fert™. *As ORIGIO® Sequential Fert™ requires CO₂ equilibration, ICSI may also be performed in a HEPES-buffered holding medium.*

It is recommended that zygotes be transferred to pre-equilibrated culture medium immediately after ICSI.

- At 16-20 hours post-insemination, check for formation of pronuclei, then carefully wash and transfer zygotes to fresh microdrops or wells of pre-equilibrated culture medium.
- Each laboratory should make its own determination of which medium to use for each particular procedure.**

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