SAGE 1-Step™

Product No.:

6701

CE
O543

Languages:

bg	cs	da	de	el
en	es	et	fr	hr
hu	is	it	kk	lt
lv	mk	nl	no	pt
ro	ru	sk	sl	sv
tr	uk			

Customer Service:

E-mail: customer.service@origio.com
Tel.: +45 46 79 02 02 Fax: +45 46 79 03 02

ORIGIO a/s
Knardrupvej 2, DK-2760 Måløv, Denmark
www.origio.com
Tel: +45 46 79 02 00 · Fax: +45 46 79 03 00

origio

a CooperSurgical Company

Ingredientia

6701 SAGE 1-Step™

Acida aminica; alanylglutaminum, Arg, Asn, Asp, Cystinum, Glu, Gly, His, Ile, Leu, Lys, Met, Phe, Pro, Ser, Thr, Trp, Tyr, Val, Acidum edeticum; Albumini humani solution: Aqua; Gentamicinum; Glucosum; Hyaluronas; KCl; KH₂PO₄; Lactas; MgSO₄; NaCl; ; NaHCO₃; Pyruvas; Phenolsulfonphthaleinum

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en - english

This product is intended for the *in vitro* culture of human embryos following fertilisation until Day 5/Day 6 of development. The medium can also be used for embryo transfer.

This product is for ART treatment, whether the cause of infertility is male or female. The product should only be used by professionals trained in ART treatment.

Products

6701 SAGE 1-Step™ with human albumin solution

Contains

Human albumin solution (HAS)
Gentamicin sulphate 10 µg/ml

Quality control testing

Sterility tested (Ph.Eur., USP)
Osmolality tested (Ph.Eur., USP)
pH tested (Ph.Eur., USP)
Endotoxin tested ≤ 0,1 EU/ml (Ph.Eur., USP)
HSA analysis (Ph.Eur., USP)
Mouse Embryo Assay (MEA) tested

Note: The results of each batch are stated on a Certificate of Analysis, which is available at www.origio.com.

Storage instructions and stability

The products are aseptically processed and supplied sterile.

Store in original container at 2-8°C, protected from light.

Do not freeze.

Discard excess (unused) media following warming.

The product is to be used within 7 days after opening.

When stored as directed by the manufacturer the product is stable until the expiry date shown on the label.

Precautions and warnings

Do not use the product if:

1. Product packaging appears damaged or if the seal is broken.
2. Expiry date has been exceeded.
3. The product becomes discoloured, cloudy, turbid, or shows any evidence of microbial contamination.

Caution: All blood products should be treated as potentially infectious. The source material used to manufacture this product was tested and found non-reactive for HBsAg and negative for Anti-HIV-1/2, HIV-1, HBV, and HCV. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated. No known test methods can offer assurances that products derived from human blood will not transmit infectious agents.

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Note: Please note the need for traceability of this product. In addition, national legal

requirements for this field may exist in your country.

Note: Only to be used in combination with other devices intended for the particular purpose.

Note: Dispose of the device in accordance with local regulations for disposal of medical devices.

Instructions for use

1. Equilibrate for a minimum of 2 hours in 5-6% CO₂ at 37 °C prior to use. It is recommended that the pH of the medium be measured under laboratory working conditions and the level of CO₂ adjusted to attain the desired pH range for optimal embryo development. The desired pH range for SAGE 1-Step™ is 7,3 ± 0,1.
2. After fertilisation, transfer zygotes to fresh pre-equilibrated SAGE 1-Step™ and return the dish to the incubator.
3. The SAGE 1-Step™ has been designed for uninterrupted culture, without media change, until the desired developmental stage of the embryo has been reached.

Embryo transfer

1. The embryos are prepared and transferred to the uterus in fresh pre-equilibrated SAGE 1-Step™.
2. Flush the transfer catheter with SAGE 1-Step™ prior to use.

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